



# OASIS | VISION®

## PRODUCT CATALOG



844.820.8940

[customerservice@oasismedical.com](mailto:customerservice@oasismedical.com)

[oasismedical.com](http://oasismedical.com)

## QUALITY ASSURANCE WORLDWIDE

The OASIS® Medical Quality System is structured to meet the requirements of the Food and Drug Administration (FDA) Quality System Regulations. OASIS® Medical is registered with the FDA as a:

- Medical Device Establishment - OASIS® Quality System complies with the Code of Federal Regulations (21 CFR §820)
- Drug Establishment - OASIS® Quality System complies with the Code of Federal Regulations (21 CFR §210/211)

The OASIS® Medical Quality Management System is certified by DQS Medizinprodukte GmbH to EN ISO 13485:2016, Medical Device Single Audit Program (MDSAP), and to EU Regulation 2017/745 (EU MDR) and EU Regulation 2023/607 concerning medical devices for applying the CE marking to products

CE Marked Product Lines



Silicone Punctum Plugs (Original Inserter REF codes only)  
Disposable Microsurgical Instruments  
HPMC Viscoelastic devices  
PVA Foam Spears  
Hydrogel Plugs  
Extended Duration Plugs (Except 180-T tapered plug)  
Microsurgical Knives  
Collagen Shields  
Iris Expander


## 3 WAYS TO ORDER:

1. Call: (844) 820-8940
2. Email: [customerservice@oasismedical.com](mailto:customerservice@oasismedical.com)
3. Register online: [www.myoasis360.com](http://www.myoasis360.com)

**Introducing MY OASIS 360** — your all-in-one, easy-to-use provider portal for everything OASIS®. Seamlessly manage your account, order and restock products, access reimbursement tools, and stay up to date with on-demand training and product protocols. MY OASIS 360 puts intuitive resources and streamlined support right at your fingertips — so you can focus on delivering the best in eye care.

### Benefits include:

- Simple, intuitive reimbursement tools
- Hassle-free product ordering and restocking
- On-demand video education for dry eye and product protocols
- Built-in communication tools to stay connected

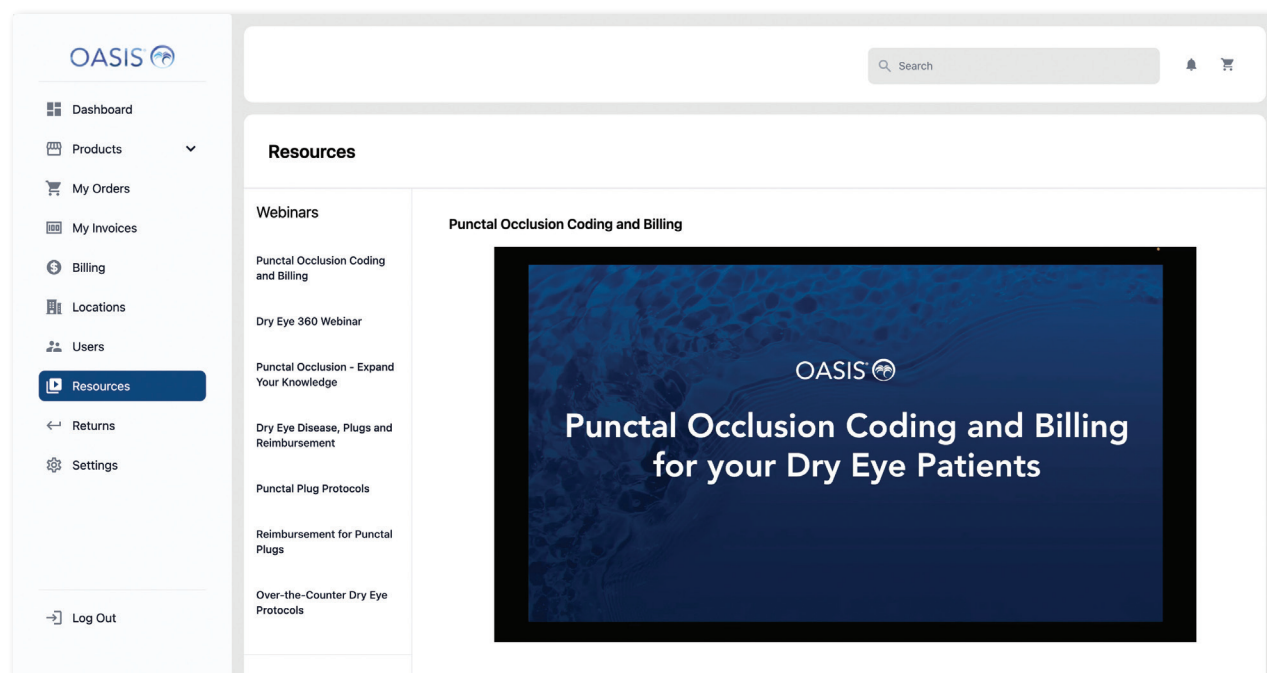

## FEATURED VIDEOS



DRY EYE 360 WEBINAR



PUNCTAL PLUG PROTOCOLS



REIMBURSEMENT FOR PUNCTAL PLUGS

Find these and more videos at: [myoasis360.com/training](http://myoasis360.com/training)

# MY OASIS™ RESOURCE CENTER

If your practice is seeking alternative ways to bring vision care solutions that can make a difference to your dry eye sufferers, then OASIS would like to introduce to your practice the MY OASIS Platform.

MY OASIS platform was developed for practices to support the expansion of their services through continuity of care. This automated platform provides a customized storefront, the ability to bridge and engage with your patient community, all while bringing the convenience of at-home delivery on recurring or non-recurring basis. Let the platform tools be your website marketing. Let your practice's key person reach new members and build your vision care/dry eye community, prevent drop off, and bring in supplemental income. MY OASIS cyber platform diversifies your outreach, which broadens your growth potential. It supports communication, connection to nurture relationships, and the ability to gain new actionable insights.


## FOR THE DOCTOR, MY OASIS IS:

- A convenient approach to customizing and delivering a daily eye care regimen with OASIS solutions.
- Access to eye care products that make a difference through a custom OASIS® store-front. Invite your members to easily place orders remotely with your unique code (recurring order option available).
- The ability to invite members to learn through the educational information offered covering lubricating eye drops, eye lid hygiene and maintenance, how to use them, and more.
- Access to tools that display your practice's growth, patient retention, and at-home care compliance.



## FOR THE CONSUMER/DRY EYE SUFFERER, MY OASIS IS:

- Well being because of access to your at-home care.
- Consumers with dry eye are seeking a vision care provider that is committed to their eye care. MY OASIS puts your practice in the palm of your patient's hand.
- Having a resource center where they can learn about dry eye and educate themselves on their treatment options.
- Convenience of ordering their doctor recommended solutions from home.
- The ease of setting their at-home care on a scheduled shipment.
- The ability to stay up to date on new products and services.

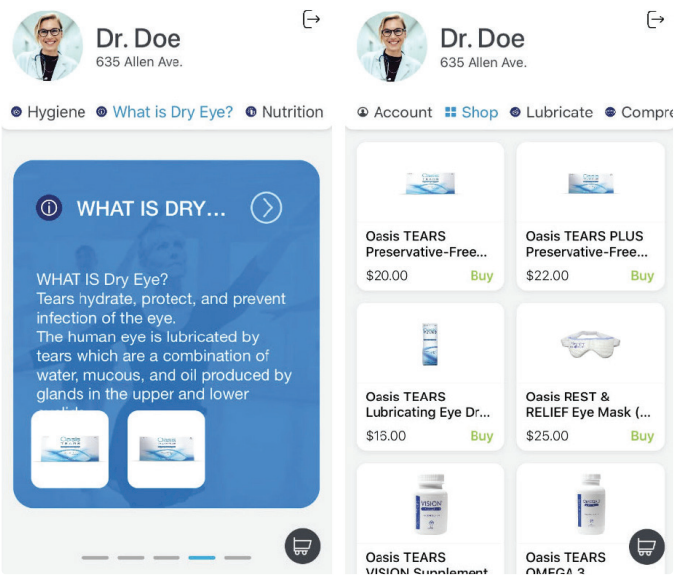




## WHAT'S NEXT? MYOASIS 2.0

We're excited to announce **myOasis 2.0**, a fully refreshed platform expected to launch in early 2026 — designed with one goal in mind: **to make it even easier to care for your patients and grow your practice.**

Building on everything you love about myOasis, version 2.0 will bring a host of new tools and enhancements, including:

- **A more intuitive provider dashboard** — Get a clearer view of patient activity, recommended products, and key performance insights, all in one place.
- **Streamlined patient product recommendations** — Quickly match patients with the right Oasis products and simplify the purchase path.
- **Improved data and insights** — Access better analytics on patient orders, engagement trends, and follow-up opportunities to strengthen your ongoing care.
- **Enhanced co-branded content and communications** — Deliver personalized reminders and recommendations that keep you top of mind with patients long after they leave your office.
- **Easier patient onboarding** — New tools to help your team introduce myOasis to patients — so they understand the benefits and are ready to order right away.
- **Seamless mobile experience** — A redesigned mobile interface that makes it easy for patients to reorder from any device, anytime.

With myOasis 2.0, you'll stay connected to your patients in a way that feels simple, supportive, and care-focused — helping you grow your business while delivering the best in eye health.

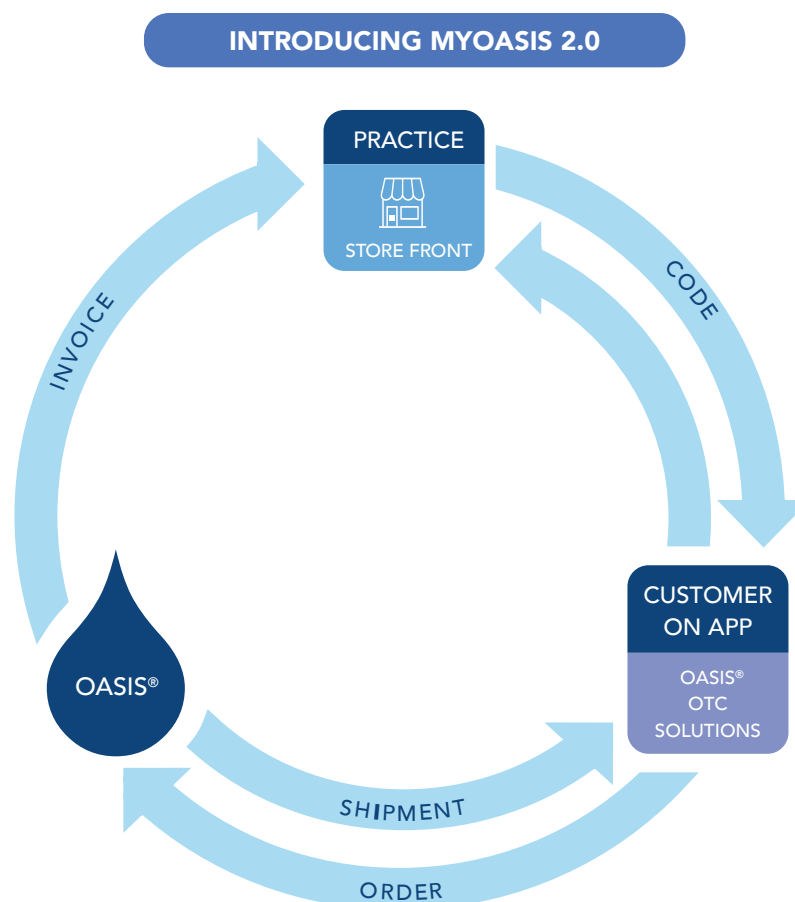

# NEW LOOK

COMING SOON











OCULAR LUBRICATION

# Oasis<sup>®</sup> TEARS<sup>®</sup>

PRESERVATIVE-FREE  
LUBRICANT EYE DROPS

VISCOADAPTIVE RELIEF  
FOR DRY EYES

30 STERILE (0.3 ML/0.01 FL OZ.)  
SINGLE USE CONTAINERS

N 42126-6100-1

# OASIS TEARS®

## LUBRICANT EYE DROPS

### RELIEF FOR DRY EYES

Oasis TEARS® Lubricant Eye Drops' proprietary formulation soothes, lubricates, and relieves burning, irritation, dryness, and other symptoms related to dry eye. Oasis TEARS® line of dry eye relief products are available through eye care professionals and not available in major retail stores. Oasis TEARS is an over-the-counter lubricant eye drop.

### DETAILS

- Dry eye relief
- Moistening, lubrication and comfort
- For use as needed
- 3 treatment options
- Single use containers or multi-use bottle

Ask how Oasis can support your practice with dry eye solutions implementation and staff training.  
NOT AVAILABLE FOR EXPORT.

REF #	PRODUCT NAME	DESCRIPTION	QTY PER CARTON
OT6500	Oasis TEARS PF Plus	Preservative-free, 0.34 fl. oz. (10ml) bottle	50 bottles
OT6400	Oasis TEARS PF	Preservative-free, 0.34 fl. oz. (10ml) bottle	20 bottles
OT6100	Oasis TEARS	Preservative-free, 30 single-use containers	45 boxes
OT6200	Oasis TEARS PLUS	Preservative-free, 30 single-use containers	45 boxes
OT6310	Oasis TEARS	With a mild preservative, 0.34 fl. oz. (10ml) bottle	20 bottles





# OASIS REST & RELIEF®

## HOT & COLD EYE MASK

COMPRESS

### RELIEF FOR DRY EYES

Oasis REST & RELIEF provides heat therapy for the relief of dry eye symptoms. Use as directed.

### PRODUCT FEATURES

- Temperature retaining beads that conform to the body in a soft knit cloth mask for maximum comfort
- Adjustable strap for comfortable compression
- Washable cover


REF #	PRODUCT NAME	DESCRIPTION	QTY PER CARTON
OM8000	Oasis REST & RELIEF EYE MASK	Eye mask for hot and cold use, 1 mask and 1 mask cover per box	24 Masks
OM8001	Oasis REST & RELIEF EYE MASK Cover	Eye mask cover, 1 mask cover	1 Mask Cover







COMPRESS

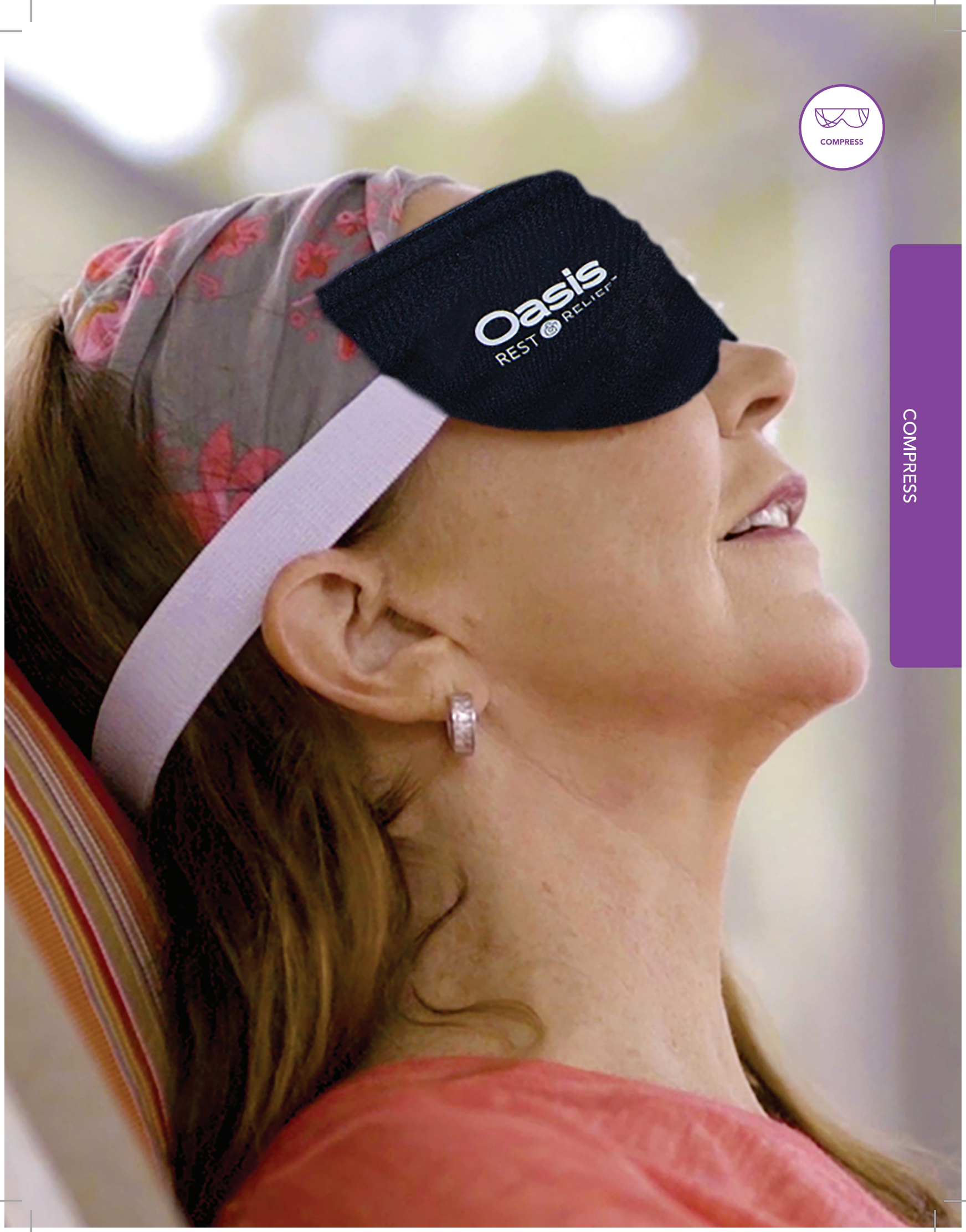





LID HYGIENE



# OASIS TEARS

# HYPOCHLOROUS®

## EYELID CLEANSING SPRAY

**RELIEF FOR DRY EYES**

Oasis TEARS HYPOCHLOROUS is a gentle, non-toxic and natural antimicrobial formula for use daily on eyelids and lashes. Oasis TEARS HYPOCHLOROUS contains Hypochlorous Acid a known antimicrobial, which serves to inhibit the growth of microorganisms. Hypochlorous Acid cleanses irritated eyelids of bioburden. This natural antimicrobial can be sprayed directly on closed eyelids to soothe them from irritation. The no-rinse spray can air-dry on to manage bioburden.

**DETAILS**

- Gentle for daily use
- .012% Hypochlorous Acid
- Natural Antimicrobial

REF #	PRODUCT NAME	DESCRIPTION	QTY PER CARTON
OT7400	Oasis TEARS HYPOCHLOROUS Spray	Eyelid and lash cleansing spray, 1.86 fl. oz. (55ml) bottle	20 bottles



LID HYGIENE

# OASIS LID & LASH<sup>®</sup>

## + TEA TREE OIL

### EYELID & LASH CLEANSING PADS

#### DAILY LID & LASH HYGIENE

Oasis TEARS Oasis LID & LASH + Tea Tree oil cleanses and hydrates eyelids and lashes with our new gentle tea tree formulation to maintain eyelid and lash appearance. Our soft pads gently remove eye debris and dust particles to cleanse delicate skin around your eyes. Leave on to air-dry smooth for a refreshing experience of clean with tea tree.

#### DETAILS

- Gentle for daily use
- Paraben Free
- 60 cleansing pads

REF #	PRODUCT NAME	DESCRIPTION	QTY PER CARTON
OT7300	Oasis TEARS Oasis LID & LASH + tea tree oil	Eyelid and lash cleansing pad, 60 pre-soaked pads	24 Jars







LID HYGIENE







NUTRITION





# OASIS TEARS OMEGA<sup>®</sup> 3

## DIETARY SUPPLEMENT

**OASIS TEARS OMEGA<sup>®</sup> 3**

Oasis TEARS OMEGA 3 uses proprietary methods to remove toxins and metals for highest purity, plus an additional process to enhance the concentration of EPA and DHA.

**DETAILS**

- Tryglyceride form
- Total DHA & EPA 1000mg per softgel
  - DHA 429mg per softgel\*\*
  - EPA 571mg per softgel\*\*
- 90 softgels = 3 softgels per serving
- Natural lemon flavor added
- No fish taste

REF #	PRODUCT NAME	DESCRIPTION	QTY PER CARTON
ON3000	Oasis TEARS OMEGA 3	Dietary Supplement	12 Bottles



NUTRITION

Oasis TEARS OMEGA 3 and Oasis TEARS VISION are free from \*gluten, \*dairy, and made with NON-GMO ingredients. Oasis TEARS OMEGA 3 and Oasis TEARS VISION are produced in a NSF, FDA Registered, and Inspected Facility. \*Oasis TEARS OMEGA 3 and Oasis TEARS VISION are produced in a facility that processes other ingredients that may contain these allergens.

# OASIS TEARS VISION®

## DIETARY SUPPLEMENT

### OASIS TEARS VISION®

Oasis TEARS VISION's patent pending formulation supports dietary supplementation for general eye health.



[VIEW THE  
WHITE PAPER](#)

### DETAILS

- Patent pending formulation
- Maqui Berry (30mg)
- Astaxanthin (6mg)
- Zeaxanthin (5mg)
- Lutein (25mg)
- DHA (200mg)
- Small softgel capsule
- 30 softgels = 1 softgel per serving


REF #	PRODUCT NAME	DESCRIPTION	QTY PER CARTON
ON3010	Oasis TEARS VISION	Dietary Supplement	12 Bottles



Oasis TEARS OMEGA 3 and Oasis TEARS VISION are free from \*gluten, \*dairy, and made with NON-GMO ingredients. Oasis TEARS OMEGA 3 and Oasis TEARS VISION are produced in a NSF, FDA Registered, and Inspected Facility. \*Oasis TEARS OMEGA 3 and Oasis TEARS VISION are produced in a facility that processes other ingredients that may contain these allergens.



NUTRITION

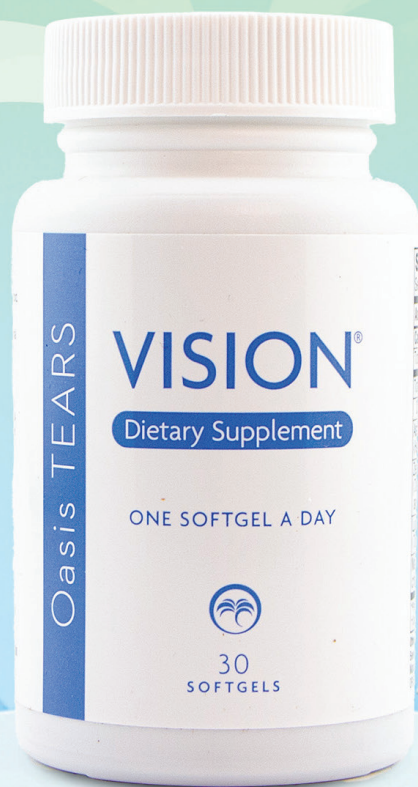



DRY EYE RELIEF KITS

OASIS DRY EYE RELIEF KIT 1

REF #: K-DEKITS-003



PRODUCT NAME	DESCRIPTION	DIMENSIONS	QTY PER BAG
Oasis TEARS PF Plus	Preservative-free eye drops		1 bottle
Oasis TEARS Oasis LID & LASH + tea tree oil	Eyelid and lash cleansing pad, 60 pre-soaked pads		1 jar
Oasis REST & RELIEF EYE MASK	Eye mask for hot and cold use		1 mask and 1 mask cover
OASIS Dry Eye Kitting Bag	Fabric Bag with Zipper	9 x 3 x 6.5	1 bag

OASIS DRY EYE RELIEF KIT 2

REF #: K-DEKITS-004



PRODUCT NAME	DESCRIPTION	DIMENSIONS	QTY PER BAG
Oasis TEARS PF Plus	Preservative-free eye drops		1 bottle
Oasis TEARS Oasis LID & LASH + tea tree oil	Eyelid and lash cleansing pad, 60 pre-soaked pads		1 jar
Oasis REST & RELIEF EYE MASK	Eye mask for hot and cold use		1 mask and 1 mask cover
Oasis TEARS OMEGA 3	Dietary Supplement, 90 Softgels		1 bottle
OASIS Dry Eye Kitting Bag	Fabric Bag with Zipper	9 x 3 x 6.5	1 bag

DRY EYE RELIEF KITS

# OASIS DRY EYE RELIEF KIT 3

REF #: K-DEKITS-005



PRODUCT NAME	DESCRIPTION	DIMENSIONS	QTY PER BAG
Oasis TEARS PF	Preservative-free eye drops		1 bottle
Oasis TEARS HYPOCHLOROUS Spray	Eyelid and lash cleansing spray		1 bottle
Oasis REST & RELIEF EYE MASK	Eye mask for hot and cold use		1 mask and 1 mask cover
OASIS Dry Eye Kitting Bag	Fabric Bag with Zipper	9 x 3 x 6.5	1 bag

# OASIS CUSTOM KITS

## CONTACT US TO CREATE A CUSTOM VISION CARE KIT

Pick from our OTC line of dry eye solutions to create your practice's custom pre-op and post-op kits using a choice of canvas bag or box with insert. Dimensions are approximate.




REF #	PRODUCT NAME	DESCRIPTION	DIMENSIONS
S7100	Oasis Dry Eye Kitting Bag	Fabric Bag with Zipper	9 x 3 x 6.5
S8100	Oasis TEARS Box	Box without Insert	11 x 8 x 3

DRY EYE RELIEF KITS



## PUNCTAL OCCLUSION

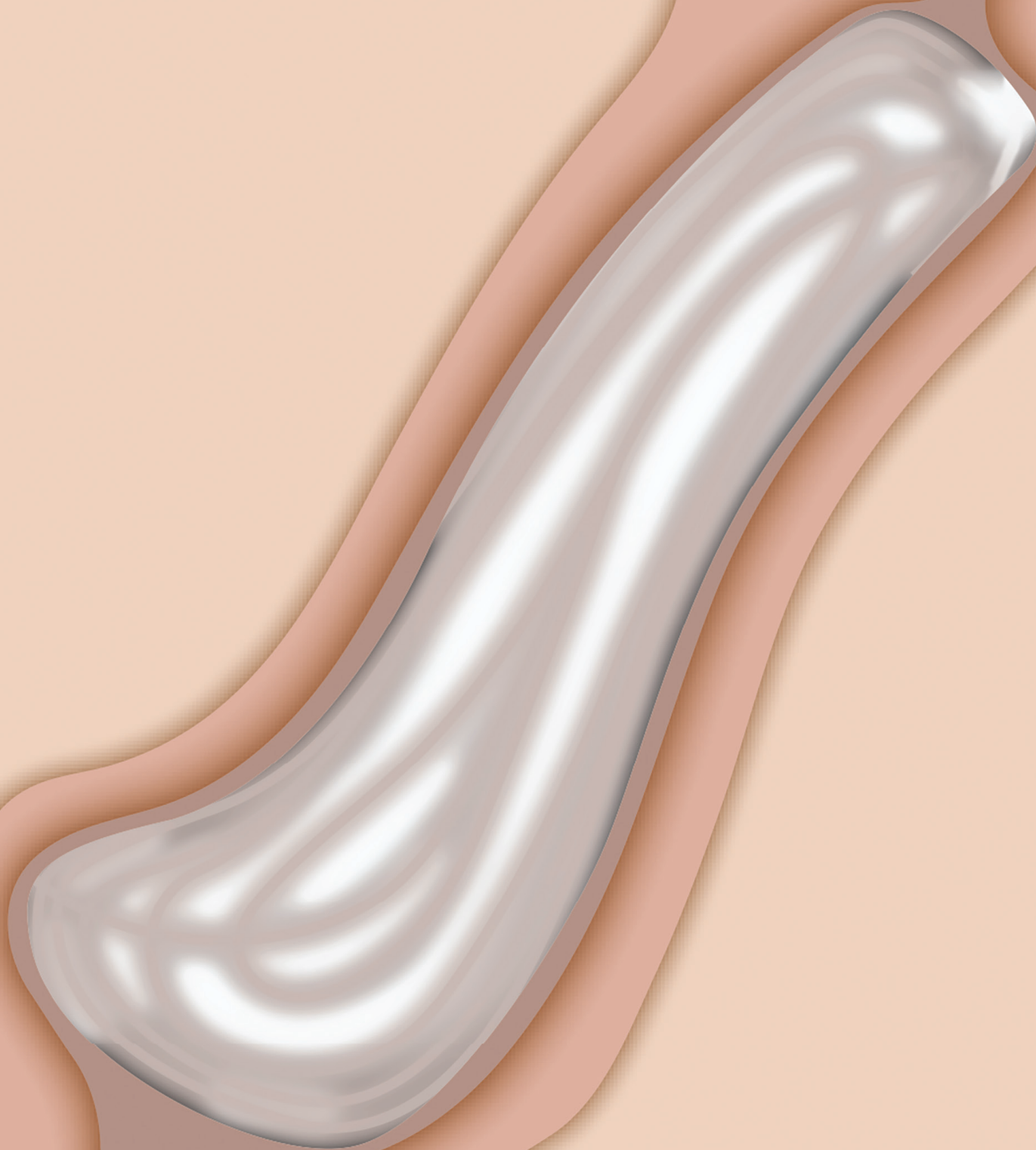



# OASIS FORM FIT<sup>®</sup>

## HYDROGEL CANALICULAR PLUGS

### LONG TERM OCCLUSION

FORM FIT is truly a one size plug solution. Once inserted into the canaliculus, FORM FIT hydrates in approximately 10 minutes. As it hydrates, FORM FIT increases in size until it completely fills the vertical canalicular cavity.


### DETAILS

- Hydrates in approximately 10 minutes
- Easily removed via irrigation of the tear duct
- Biocompatible, no foreign body sensation
- Sterile
- Conveniently pre-loaded in inserter
- 0.3mm, one size fits all

REF #	PRODUCT NAME	QTY PER BOX	DIAMETER
6303	FORM FIT Hydrogel Canalicular Plug	10pk	0.3mm
6303-T	FORM FIT Hydrogel Canalicular Plug	2pk	0.3mm

### HOW IT WORKS

FORM FIT is made of a safe and biocompatible hydrogel material that expands into a soft, pliable, gelatinous material when it contacts tear film. FORM FIT securely conforms to the vertical canaliculus so there is no need for a cap on the end of the plug. This greatly increases patient comfort and decreases foreign body sensation.

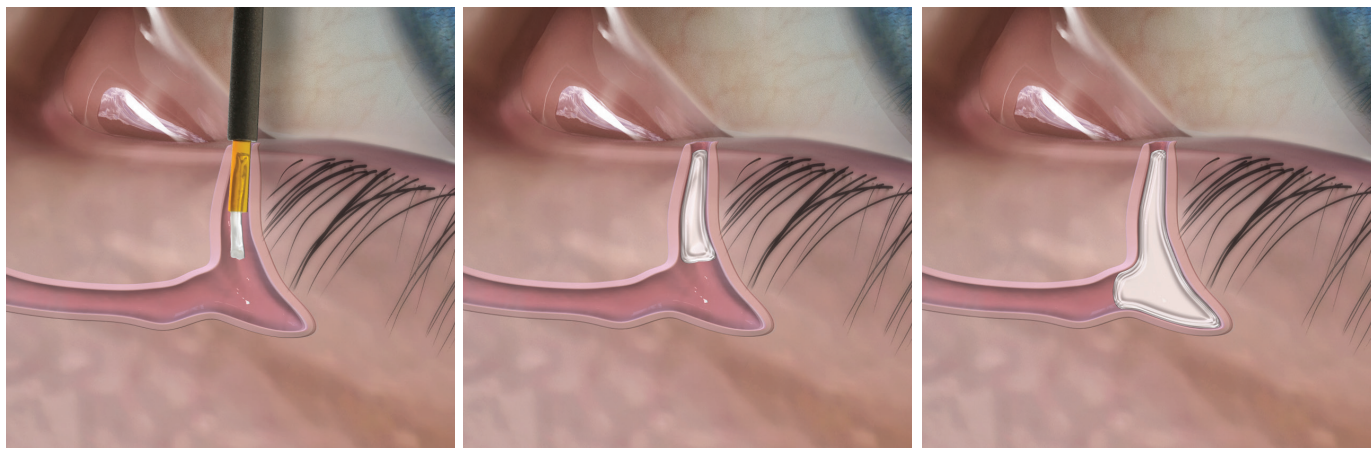

PUNCTAL OCCLUSION

# OASIS SOFT PLUG®


## SILICONE PUNCTUM PLUGS

### LONG TERM OCCLUSION

The distinctive low-profile cap combined with a softer, flexible silicone maximizes patient comfort. Now available with two inserter options. SOFT PLUG Silicone Punctum Plugs are removable with forceps per instruction for use.

### DETAILS

- NEW Pen-Style Inserter with dilator tip - 2pks, 6pks
- Original Inserter with dilator tip - 2pks, 6pks, 20pks
- Sterile, preloaded on inserter in 2-pack or 6-pack dispenser box
- Low-profile cap intended to reduce dislocation
- Economy pack includes 20 plugs and two non-sterile inserters. Holders equipped with an alignment slot for quick loading

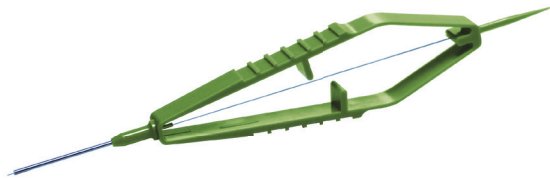

Original Inserter




Pen-Style Inserter

PUNCTAL OCCLUSION

ORIGINAL INSERTER REF #		PRODUCT NAME	QTY PER BOX	DIAMETER
6610		SOFT PLUG Silicone, Preloaded, Micro	2pk	0.4mm
6611		SOFT PLUG Silicone, Preloaded, Mini	2pk	0.5mm
6612		SOFT PLUG Silicone, Preloaded, Petite	2pk	0.6mm
6613		SOFT PLUG Silicone, Preloaded, Small	2pk	0.7mm
6614		SOFT PLUG Silicone, Preloaded, Medium	2pk	0.8mm
ORIGINAL INSERTER REF #	PEN-STYLE INSERTER REF #	PRODUCT NAME	QTY PER BOX	DIAMETER
6610-D	NEW! 7710-D	SOFT PLUG Silicone, Preloaded, Micro	6pk	0.4mm
6611-D	NEW! 7711-D	SOFT PLUG Silicone, Preloaded, Mini	6pk	0.5mm
6612-D	NEW! 7712-D	SOFT PLUG Silicone, Preloaded, Petite	6pk	0.6mm
6613-D	NEW! 7713-D	SOFT PLUG Silicone, Preloaded, Small	6pk	0.7mm
6614-D	NEW! 7714-D	SOFT PLUG Silicone, Preloaded, Medium	6pk	0.8mm
ORIGINAL INSERTER REF #		PRODUCT NAME	QTY PER BOX	DIAMETER
6610-EP		SOFT PLUG Silicone, Economy Pack, Micro	20pk	0.4mm
6611-EP		SOFT PLUG Silicone, Economy Pack, Mini	20pk	0.5mm
6612-EP		SOFT PLUG Silicone, Economy Pack, Petite	20pk	0.6mm
6613-EP		SOFT PLUG Silicone, Economy Pack, Small	20pk	0.7mm
6614-EP		SOFT PLUG Silicone, Economy Pack, Medium	20pk	0.8mm



PUNCTAL OCCLUSION

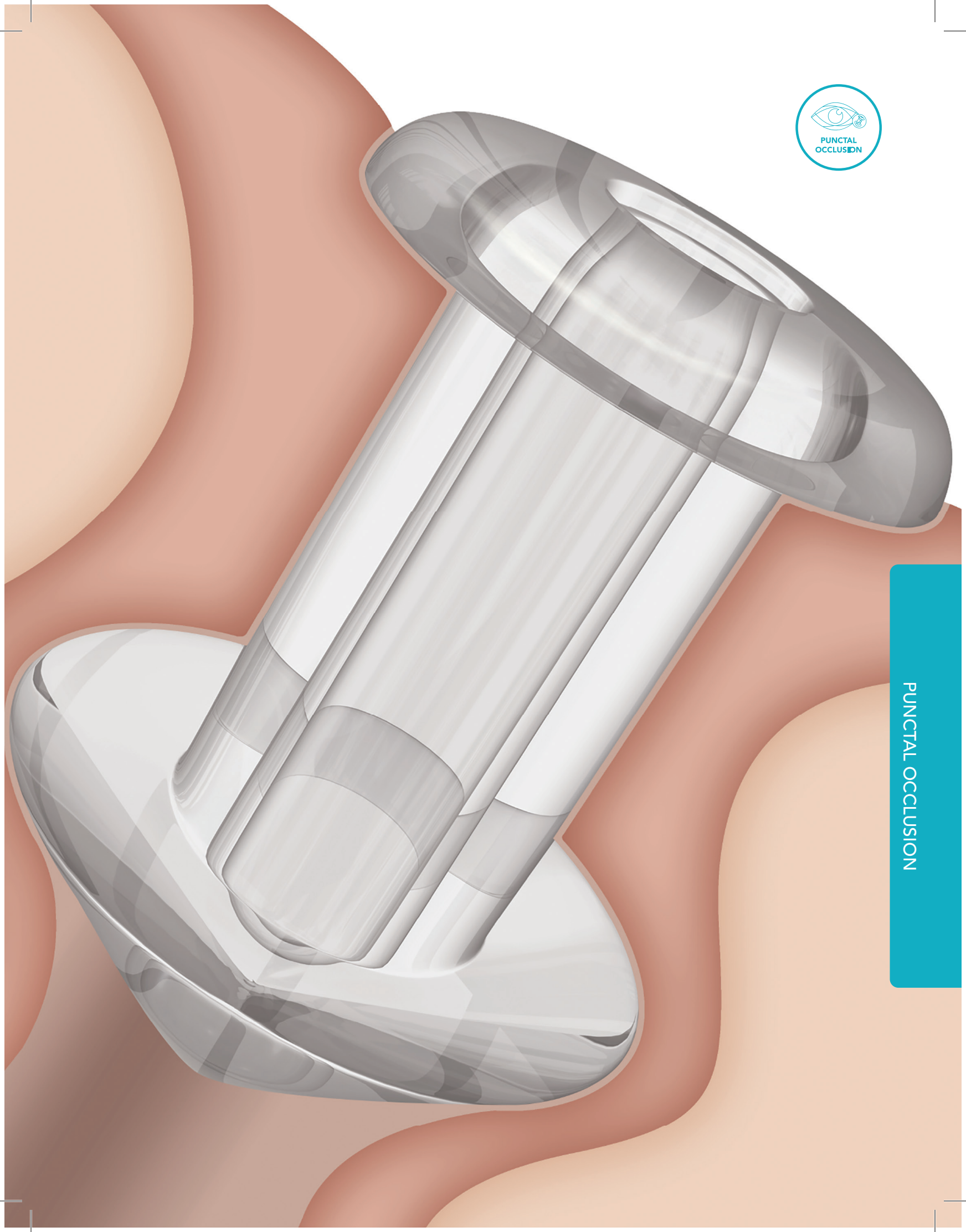

# OASIS SOFT PLUG®

## SILICONE PUNCTUM PLUG, FLOW CONTROL

### LONG TERM OCCLUSION WITH PARTIAL DRAINAGE


The distinctive low-profile cap combined with a softer, flexible silicone maximizes patient comfort. The inserter minimizes potential for accidental release of the plug. Plug is designed with a center channel allowing for partial tear flow. SOFT PLUG Flow Control Plugs are removable with forceps (per instruction for use).

### DETAILS


- Silicone
- Sterile, preloaded on inserter
- For partial occlusion with limited tear drainage

REF #	PRODUCT NAME	QTY PER BOX	DIAMETER
6612-FC	SOFT PLUG Flow Control, Preloaded, Petite	2pk	0.6mm
6613-FC	SOFT PLUG Flow Control, Preloaded, Small	2pk	0.7mm
6614-FC	SOFT PLUG Flow Control, Preloaded, Medium	2pk	0.8mm

PUNCTAL OCCLUSION

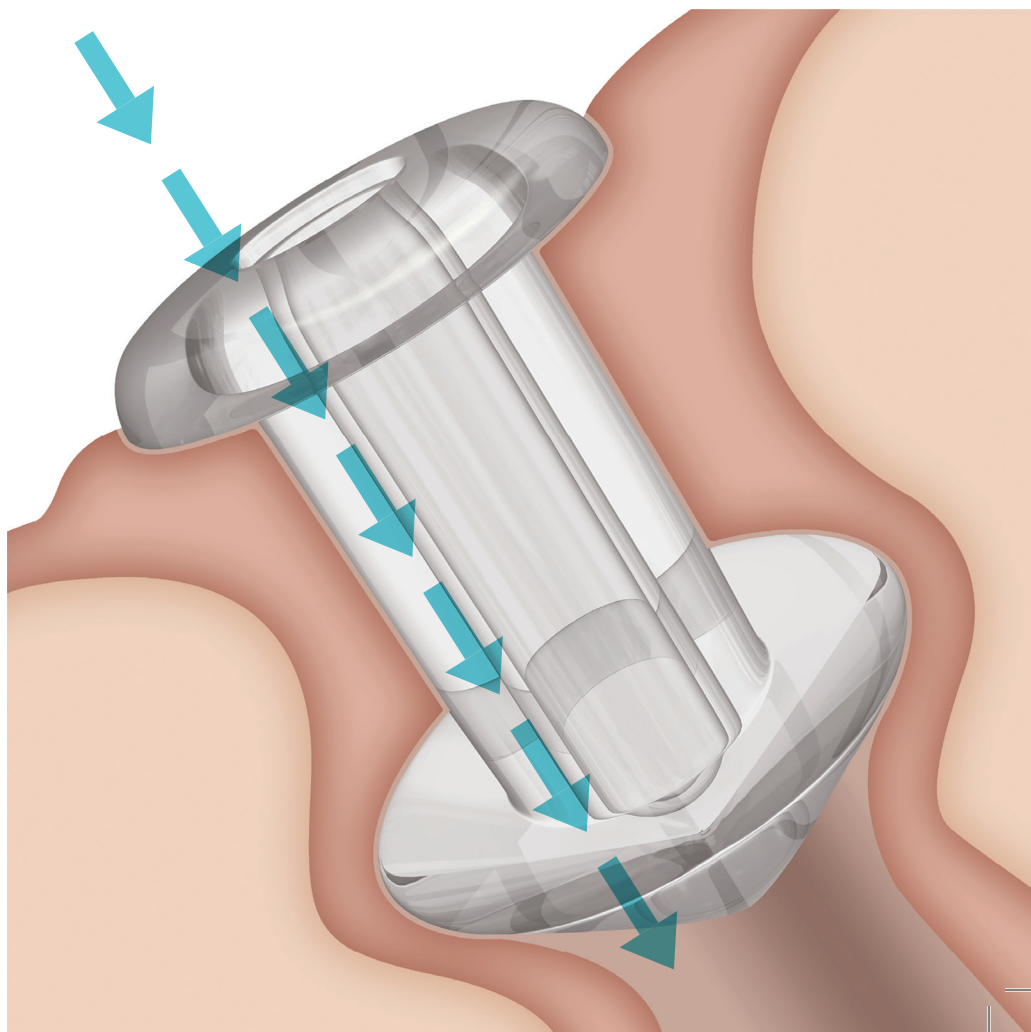



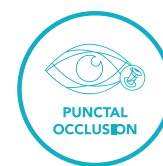

# OASIS SOFT PLUG<sup>®</sup>

## COLLAGEN INTRACANALICULAR PLUGS

### TEMPORARY OCCLUSION

The Collagen Plugs have a smooth proprietary finish that simplifies insertion of the plug. Collagen Plugs are ideal for short-term, diagnostic, and post-surgical occlusion.

### DETAILS

- Degradable material lasts 2-5 days
- Proprietary finish simplifies insertion
- Evaluate patient acceptance
- 2mm length
- Sterile
- Insert with forcep

REF #	PRODUCT NAME	QTY PER BOX	DIAMETER
6502	SOFT PLUG Collagen Intracanalicular Plug	60pk	0.2mm
6503	SOFT PLUG Collagen Intracanalicular Plug	60pk	0.3mm
6504	SOFT PLUG Collagen Intracanalicular Plug	60pk	0.4mm

# OASIS SOFT PLUG®

## EXTENDED DURATION PLUGS

### MEDIUM TERM OCCLUSION FOR UP TO 3 OR 6 MONTHS



SOFT PLUG Extended Duration Punctum Plugs are degradable canalicular occluders that are an effective option for transient dry eye symptoms before or after surgery, seasonally, etc. and for patients seeking to experience non-permanent options of punctal occlusion.

### DETAILS


- Pre and post-surgical use
- Sterile
- Insert with forceps
- 2mm length

REF #	PRODUCT NAME	QTY PER BOX	DIAMETER
<b>SOFT PLUG® EXTENDED DURATION 180-T TAPERED CANALICULAR PLUG</b>			
6210	SOFT PLUG Extended Duration 180-T Tapered Plug	40pk	Tapered 0.6mm - 0.25mm
<b>SOFT PLUG® EXTENDED DURATION 180 CANALICULAR PLUG</b>			
6203-A	SOFT PLUG Extended Duration 180 Canalicular Plug	20pk	0.3mm
6204-A	SOFT PLUG Extended Duration 180 Canalicular Plug	20pk	0.4mm
6205-A	SOFT PLUG Extended Duration 180 Canalicular Plug	20pk	0.5mm
6203	SOFT PLUG Extended Duration 180 Canalicular Plug	40pk	0.3mm
6204	SOFT PLUG Extended Duration 180 Canalicular Plug	40pk	0.4mm
6205	SOFT PLUG Extended Duration 180 Canalicular Plug	40pk	0.5mm
6200V	SOFT PLUG Extended Duration 180 Canalicular Plug Variety Pack	20pk	3 pair 0.3mm, 4 pair 0.4mm, 3 pair 0.5mm
<b>SOFT PLUG® EXTENDED DURATION PLUG</b>			
6402-A	SOFT PLUG Extended Duration Plug	20pk	0.2mm
6403-A	SOFT PLUG Extended Duration Plug	20pk	0.3mm
6404-A	SOFT PLUG Extended Duration Plug	20pk	0.4mm
6405-A	SOFT PLUG Extended Duration Plug	20pk	0.5mm
6402	SOFT PLUG Extended Duration Plug	40pk	0.2mm
6403	SOFT PLUG Extended Duration Plug	40pk	0.3mm
6404	SOFT PLUG Extended Duration Plug	40pk	0.4mm
6405	SOFT PLUG Extended Duration Plug	40pk	0.5mm
6400V	SOFT PLUG Extended Duration Plug, Variety Pack	20pk	1 pair 0.2mm, 3 pair 0.3mm, 4 pair 0.4mm, 2 pair 0.5mm




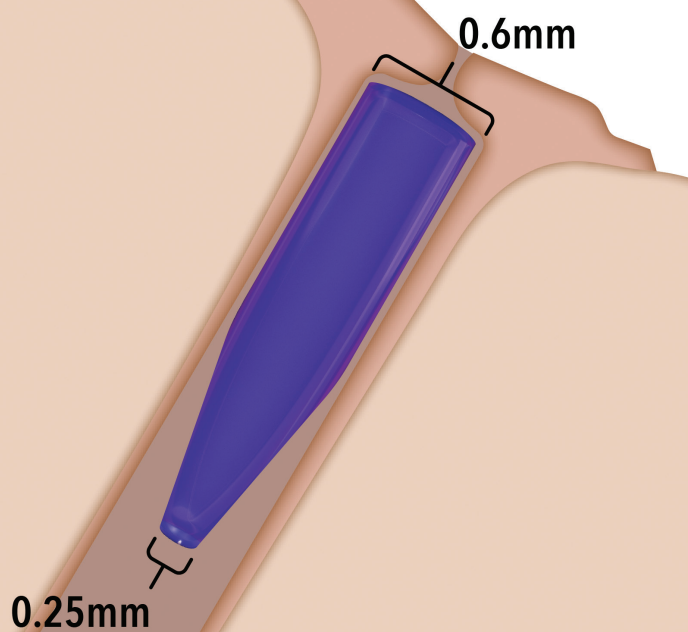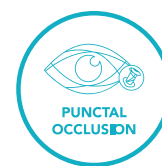

**Soft Plug®**  
**Extended Duration 180-T**  
Tapered Canalicular Plug  
Up to 180 days

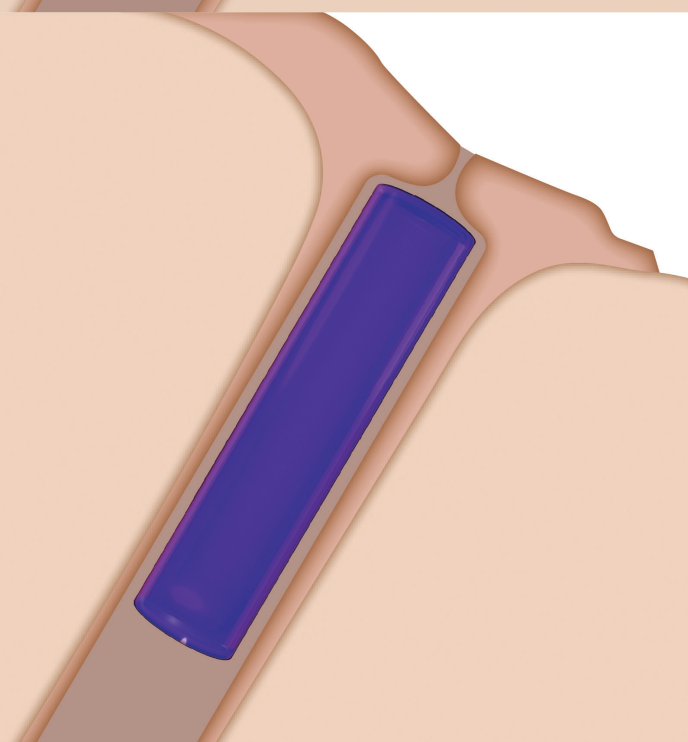


**Soft Plug®**  
**Extended Duration 180**  
Canalicular Plug  
Up to 180 days

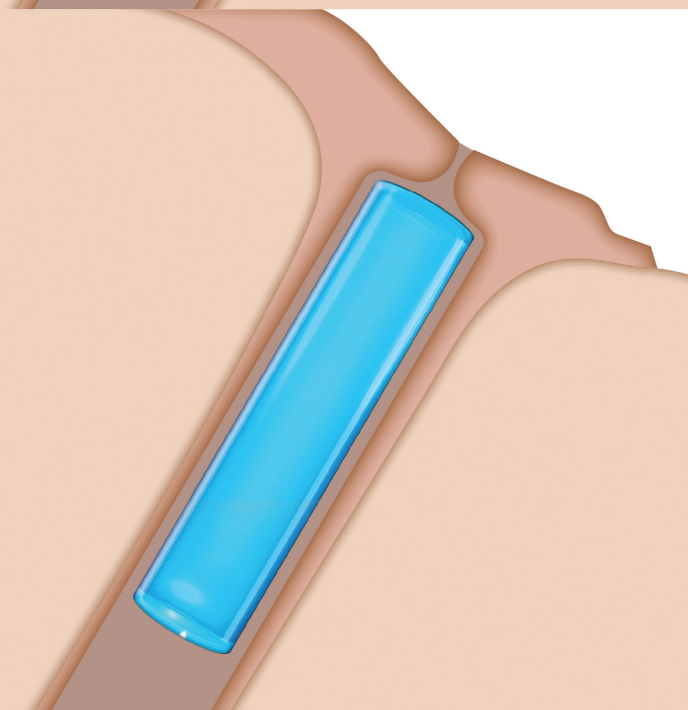

**Soft Plug®**  
**Extended Duration Plug**  
Up to 90 days

# ACCESSORIES & DRY EYE DIAGNOSTICS

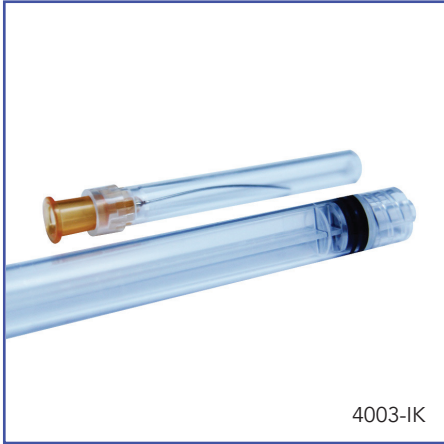

REF #	PRODUCT NAME	DESCRIPTION	QTY PER BOX
<b>FORCEPS</b>			
6643	Straight Jewelers Forceps	Straight #3	1
6647	Curved Jewelers Forceps	Curved #7	1
6648	Grooved Forceps	For the insertion of Extended Duration & Collagen Plugs	1
<b>LACRIMAL INSTRUMENTS</b>			
6640	Castroviejo Lacrimal Dilator	Double End	1
4003-IK	Lacrimal Irrigation Kit	One 3mL syringe and one 25g x 1 1/8" lacrimal cannula, curved, open end	10
6001	Lacrimal Intubation Set	23g	3
6004	Lacrimal Intubation Set with Retrieval Device	23g	3
6004-27	Lacrimal Intubation Kit	27g	3
<b>TEST STRIPS</b>			
6512	Tear Flo Test Strips	Sterile strip	100



6625



6627



6630


REF #	PRODUCT NAME	DESCRIPTION	QTY PER BOX
<b>PUNCTAL PLUG GAUGES</b>			
6625	SOFT PLUG Punctal Plug Gauge	0.5mm and 0.6mm tips	1
6627	SOFT PLUG Punctal Plug Gauge	0.7mm and 0.8mm tips	1
6630	SOFT PLUG Punctal Plug Gauge Set	Includes both instruments and sterilization tray	1 set

## FITOVER SUNGLASSES

### DETAILS

- Comfortable fit
- Lightweight frame
- Polycarbonate lenses
- Provides 48% more protection from sunlight than ordinary sunglasses
- 100% UVA/UVB protection

REF #	DESCRIPTION
OM8100	OASIS Cataract Glasses

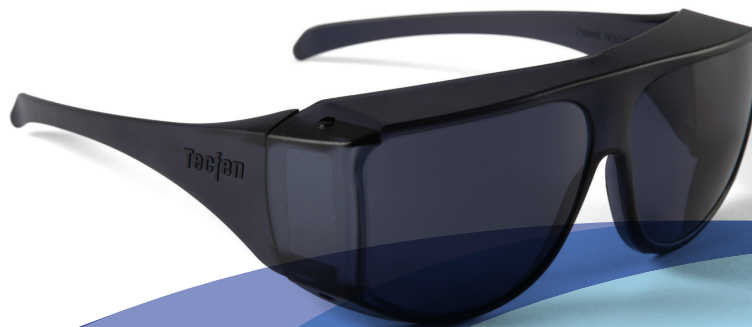



# MARKETING MATERIALS

OASIS® offers educational materials in the form of brochures, displays, posters, and videos for use in your practice and to distribute to your patients.


To view and download digital versions of all resources, go to Practice & Patient Materials at [oasismedical.com](https://oasismedical.com)

## CATALOGS

REF #	NAME
LIT-USDE-CAT	Dry Eye Catalog
LIT-SRG-CAT	Surgical Catalog



LIT-USDE-CAT

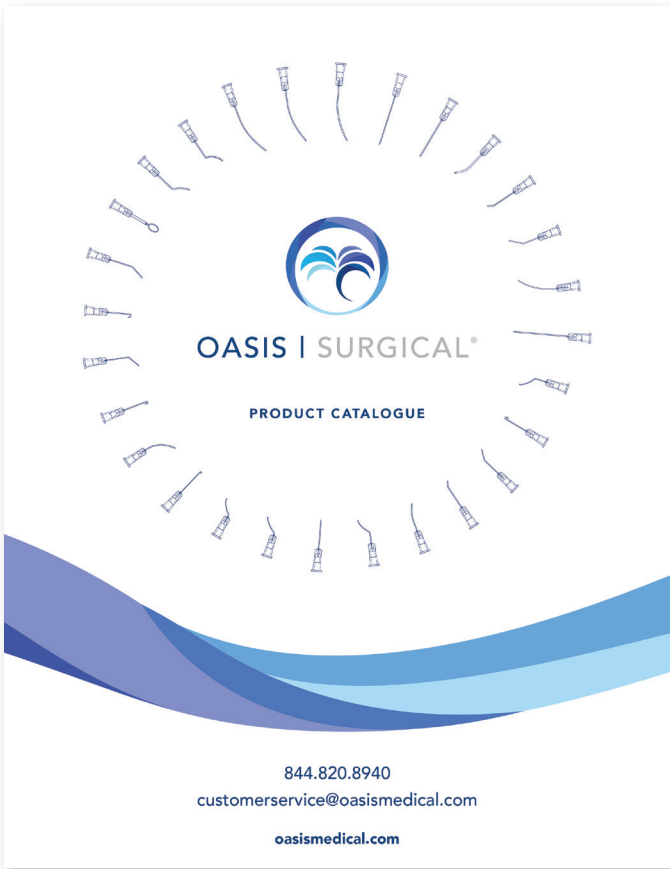

LIT-SRG-CAT

# Instant Relief of DRY EYE SYMPTOMS




## Oasis TEARS®

6544C


### ABOUT TEARS

Tears are essential for keeping your eyes healthy and your vision clear. Tears lubricate your eyes and protect the cornea (outer surface). A lack of tears, improper lubrication or tears draining too quickly can lead to a condition called Dry Eye Syndrome. If not treated, dry eye causes physical discomfort and can affect the quality of your vision.

Tears are produced by the lacrimal glands in the upper and lower eyelids and drain from your eyes through the punctum (tear ducts).

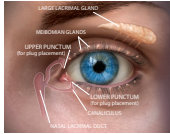

**TEAR FILM is made up of 3 layers:**

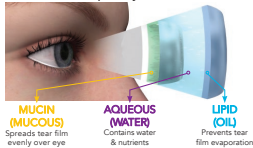

**Normal tear film**  
Lipids regulate the evaporation of the water layer. The volume and quality of the tear film on the surface of the eye is critical to maintain ocular surface health, duration of comfort and quality of vision.

**Abnormal tear film**  
For a dry eye sufferer an abnormal tear film develops with a reduction of the lipids, water and mucin layers. Areas of dryness form causing surface damage to the eye.

### HAVE DRY EYE SYMPTOMS?


It is important that you discuss your symptoms with your Eye Care Professional. Most dry eye symptoms will remain without treatment and affect your quality of vision. Your Eye Care Professional can perform a simple test to determine the severity of your Dry Eye and recommend the best method of treatment.

### TREATMENT - OASIS TEARS

Natural tears can be enhanced by using eye drops called ARTIFICIAL TEARS. They supplement the moisture that is missing and lubricate the eyes to temporarily relieve Dry Eye symptoms.

It is important to use a preservative-free drop such as Oasis TEARS®. Most artificial tears contain preservatives which can damage the outer layer of the cornea when using more than 4-6 drops per day.


Oasis TEARS® are a proprietary formulation which keep the tears on the eye surface.

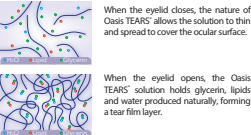

When the eyelid closes, the nature of Oasis TEARS® allows the solution to thin and spread to cover the ocular surface.

When the eyelid opens, the Oasis TEARS® solution holds glycerin, lipids and water produced naturally, forming a tear film layer.

### HOW GLYCERIN IN OASIS TEARS WORKS

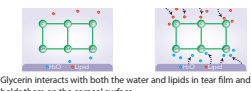

Glycerin interacts with both the water and lipids in tear film and holds them on the corneal surface.

### THE PRODUCTS


#### Oasis TEARS®

Oasis TEARS® products give your dry eyes soothing comfort by moistening and lubricating ocular tissue.

Oasis TEARS® and Oasis TEARS® PLUS are supplied in convenient "take with you" disposable containers. Oasis TEARS® (multidose) and Oasis TEARS® PF come in a 10ml bottle.

Four treatment options offer Dry Eye relief for mild to severe symptoms. Your Eye Care Professional can recommend the appropriate Oasis TEARS® product for your dry eye condition.

- Moistening, lubrication, and comfort
- Instant relief of symptoms
- No blurred vision or foreign body sensation




Description
<b>Oasis TEARS®</b> Preservative-free Lubricating Eye Drops Recommended for mild to moderate symptoms; 30 disposable containers per box
<b>Oasis TEARS® PLUS</b> Preservative-free Lubricating Eye Drops Recommended for moderate to severe symptoms; 30 disposable containers per box
<b>Oasis TEARS® PF</b> Preservative-free Lubricating Eye Drops Recommended for mild to moderate symptoms; 10ml multidose bottle
<b>Oasis TEARS® (multidose)</b> Lubricating Eye Drops Recommended for mild symptoms; 10ml multidose bottle

# Understanding PUNCTAL PLUGS for Dry Eye Relief



LIT-PP-PB

### WHAT IS DRY EYE?



Tears hydrate, protect, and prevent infection of the eye.

The human eye is lubricated by tears which are a combination of water, oil, and mucous produced by glands in the upper and lower eyelids.

A lack of any of these components results in a condition known as **Dry Eye Disease (DED)**.

— 2 Common Types of Dry Eye —

- » **Evaporative** - Lack of lipids in tears and tear film
- » **Aqueous** - Lack of water in tears and tear film

### DIAGNOSTICS

Dry eye patients are unique in that their individual case could be related to any combination and number of direct and underlying causes. Helping your eye doctor to narrow down if your dry eye is evaporative or aqueous would be the first step.

If your eye doctor offers a dry eye survey, complete it. Your answers are 50% of the work to determine your diagnosis. Your eye doctor can then determine the tests they may want to run. Here are a few types:

**Staining tests** – lissamine green, rose bengal and fluorescein staining show where the mucin is no longer protecting the surface of your eye.

**TBUT (Tear Break Up Time) know your TBUT.**  
This is the time between your last blink and the first spot on your eye to dry under what is called a slit lamp. This is usually performed with a drop of fluorescein dye to clearly identify the dry spot.


**Paper strip and thread tests** measure your tear wetting & tear volume.

**Other testing** – hand-held tear readers.

### WHAT ARE PUNCTAL PLUGS?

Once your eye doctor understands what may be triggering your dry eye symptoms, he or she can consider any combination of treatments for effective relief based on your individual dry eye diagnosis. One treatment option could be punctal plugs. Punctal plugs are a dry eye treatment that can be used alone or in combination with a preservative-free lubricant drop. They are smaller than a grain of rice and are placed in the punctum. Tears drain from your eyes through what is called the punctum and the canal below it called the canaliculus. Punctal plugs comfortably seal the drain, or punctum. This allows your own tears to remain on your eyes for a longer period of time. It is a simple non-invasive procedure done in the eye care provider's practice.


*Here are a few punctal plug possibilities to consider asking your doctor about >>>*

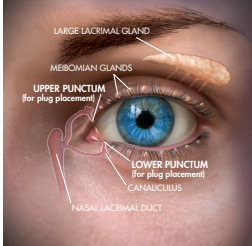

### PUNCTAL PLUGS

#### Short Term


There are short-term punctal plugs that dissolve in up to 3-5 days.




**SOFT PLUG®**  
COLLAGEN INTRACANALICULAR PLUGS

#### Medium Term

Medium term, or extended duration punctal plugs dissolve in up to 3 months, or up to 6 months.



**SOFT PLUG®**  
EXTENDED DURATION PLUGS




**SOFT PLUG®**  
EXTENDED DURATION 180 CANALICULAR PLUGS


These dissolvable plugs sit inside the canaliculus below the punctal opening and therefore are not visible. These are often called canalicular or intracanalicular plugs.

#### Long Term

If a patient prefers that their long-term plugs not be visible, then consider intracanalicular plugs made from hydrogel, a soft gelatinous material. These sit within the canaliculus clear from view.



**FORM FIT®**  
HYDROGEL CANALICULAR PLUG



**SOFT PLUG®**  
SILICONE PUNCTUM PLUGS


The traditional long term silicone punctal plug has a cap and anchor design. The silicone plugs have a slightly visible cap exposed over the punctal opening and an anchor that sits within the canaliculus to prevent, or reduce, the draining of tears.

Punctal plugs come in different sizes so it takes a few minutes for sizing and placement. The long-term silicone and hydrogel plugs are easily removed if necessary.

# BROCHURES

REF #	NAME
6544C	Oasis TEARS Brochure
LIT-PP-PB	Punctal Plug Brochure

MARKETING MATERIALS

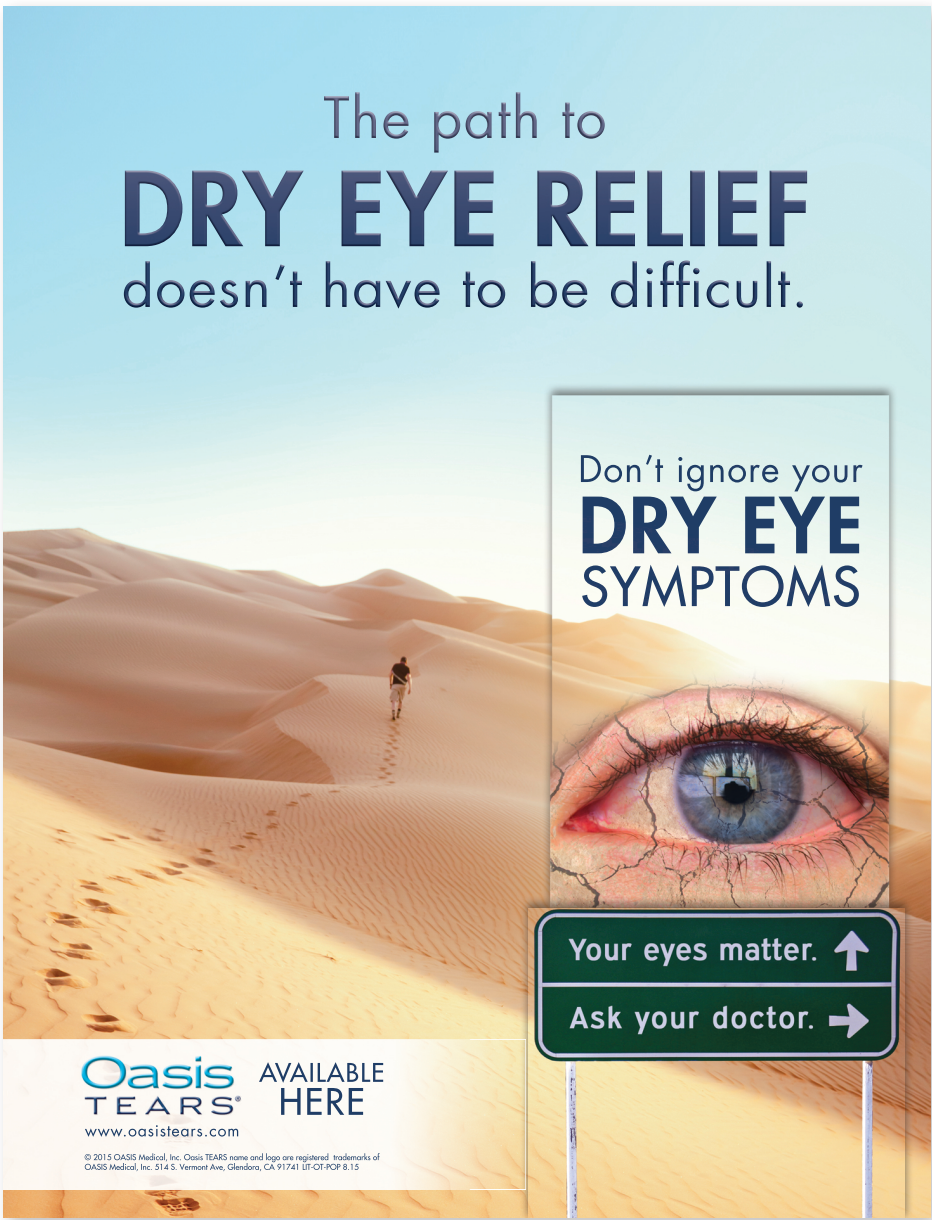

LIT-OT-POP

POINT-OF-PURCHASE DISPLAY

REF #	NAME
LIT-OT-POP	Oasis TEARS Point-of-Purchase Display
LIT-DEP-PB	Dry Eye Products Patient Brochure (English)
LIT-DEP-PBS	Dry Eye Products Patient Brochure (Spanish)
LIT-DEP-PBC	Dry Eye Products Patient Brochure (Traditional Chinese)



## CHOOSE FROM:

English  
Spanish  
Chinese (Traditional)



LIT-DEP-PB (English)

Once your eye doctor understands what may be triggering your dry eye symptoms, he or she can consider any combination of treatments for effective relief based on your individual dry eye diagnosis. Here are a few possibilities to consider asking your doctor about:

### DAILY EYELID & LASH HYGIENE

Start an eye care regimen. Daily hygiene not only applies to your mouth; it also applies to your eyes. Take care of your eyelids and the delicate skin around them by using a hydrating gel cleanser daily. Lids, lashes, and skin are moisturized using gentle cleansers like the eyelid cleansers from OASIS® Medical. Ask your eye doctor where eyelid hygiene may be a benefit.

Oasis LID  
& LASH®



&



Oasis LID  
& LASH®  
+ Tea Tree oil

**TIP:** Plan a daily eyecare regimen with your doctor

### WARM COMPRESS

Warm compresses bring relief to your dry eye because of the loosening of the oils in the Meibomian glands located in and around your lids. These glands are what produce the oils (lipids) that get released with each blink and blanket the surface of your tear film. These oils also help to retain the water layer and prevent tear evaporation. Some doctors will recommend holding a warm wash cloth to your closed eye lids, however it may be that the heat doesn't last long enough. There are compresses or soft cloth eye mask filled with beads that conform to your eyes and orbital bones and hold heat along with compression. These eye masks can be used per your doctor's recommendation.

Oasis REST & RELIEF® Eye Mask  
(hot & cold)



### PUNCTAL PLUGS

Tears drain from your eyes through what is called the puncta and the canal below it called the canaliculus. Punctal plugs comfortably seal the drain, or puncta. This allows your own tears to remain on your eyes for a longer period of time.

#### Short Term

There are short-term punctal plugs that dissolve in 3-5 days.



SOFT PLUG®  
COLLAGEN  
PLUG

#### Medium Term

Medium term or extended duration Punctal Plugs dissolve in up to 3 months or up to 6 months.



SOFT PLUG®  
EXTENDED  
DURATION  
PLUG



SOFT PLUG®  
EXTENDED  
DURATION  
180 PLUG

These dissolvable plugs sit inside the canaliculus and therefore are not visible.



#### Long Term

If a patient prefers that their long-term plugs not be visible, then consider plugs made from hydrogel, a soft gelatinous material. These sit within the canaliculus clear from view.



FORM FIT®  
HYDROGEL  
INTRACANALICULAR  
PLUG

The traditional long term silicone punctal plug has a cap and anchor design. The silicone plugs have a slightly visible cap exposed over the puncta and an anchor that sits within the canaliculus to prevent, or reduce, the draining of tears.



SOFT PLUG®  
SILICONE  
PUNCTAL  
PLUG

Punctal plugs come in different sizes so it takes a few minutes for placement and they are easily removed if necessary.

### PRESERVATIVE-FREE LUBRICANT EYE DROPS

Lubricant eye drops are artificial tears that replace the moisture that is lost on your eyes surface. The viscosity of artificial tears such as Oasis TEARS® PLUS and Oasis TEARS® lubricate the eyes to relieve the gritty dry sensations that cause irritation. Doctors often recommend preservative-free artificial tears such as these. Oasis TEARS® can be used as needed to relieve dry eye symptoms.



Oasis  
TEARS®  
PLUS  
SINGLE-USE




Oasis  
TEARS®  
SINGLE-USE

MARKETING MATERIALS

DOCTOR  
RECOMMENDATION PADS


REF #	NAME
LIT-OTRX-NB	Oasis TEARS Doctor Recommendation Pad, without OASIS Contact Information
LIT-OTRX-W	Oasis TEARS Doctor Recommendation Pad, with OASIS Contact Information

EYE CARE REGIMEN

☐


Oasis TEARS®  
Preservative-free Vials

Use: \_\_\_\_\_

☐


Oasis TEARS®  
Bottle

Use: \_\_\_\_\_

☐


Oasis REST & RELIEF®  
Hot & Cold Eye Mask

Use: \_\_\_\_\_

☐


Oasis TEARS® PLUS  
Preservative-free Vials

Use: \_\_\_\_\_

☐


Oasis TEARS® PF  
Preservative-free Bottle

Use: \_\_\_\_\_

☐


Oasis TEARS Hypochlorous™  
Eyelid Cleansing Spray

Use: \_\_\_\_\_

☐


Oasis TEARS OMEGA 3™  
Bottle

Use: \_\_\_\_\_

☐

Oasis TEARS VISION™  
Bottle

Use: \_\_\_\_\_

☐

Oasis LID & LASH® + Tea Tree Oil  
Eyelid & Lash Cleansing Pads

Use: \_\_\_\_\_


Dr. \_\_\_\_\_

Comments: \_\_\_\_\_

LITOTRX-NB Rev5 08.2020


LIT-OTRX-NB

EYE CARE REGIMEN

☐


Oasis TEARS®  
Preservative-free Vials

Use: \_\_\_\_\_

☐


Oasis TEARS®  
Bottle

Use: \_\_\_\_\_

☐


Oasis REST & RELIEF®  
Eye Mask

Use: \_\_\_\_\_

☐


Oasis TEARS® PLUS  
Preservative-free Vials

Use: \_\_\_\_\_

☐

Oasis TEARS® PF  
Preservative-free Bottle

Use: \_\_\_\_\_

☐

Oasis LID & LASH®  
Tea Tree Oil  
Cleansing Pads

Use: \_\_\_\_\_

Dr. \_\_\_\_\_

Comments: \_\_\_\_\_

OASIS Customer Service: (844) 820-8940 • www.oasistears.com • LITOTRX-W Rev4 03.2019

LIT-OTRX-W

WHAT IS  
DRY EYE?

SYMPTOMS

- Dry sensation
- Scratchy, gritty feeling
- Burning or stinging
- Itching
- Fluctuating or blurry vision
- Excess tearing (watery eyes)
- Mucous discharge
- Irritation from wind or smoke
- Redness
- Tired eyes
- Light sensitivity
- Contact lens discomfort
- Contact lens solution sensitivity
- Lid infections/sties
- Eyelids stuck together at awakening


LIT-DES-WC 05.15

LIT-DES-WC


MARKETING MATERIALS

36 OASIS 844 820-8940

# Punctal Occlusion for DRY EYE

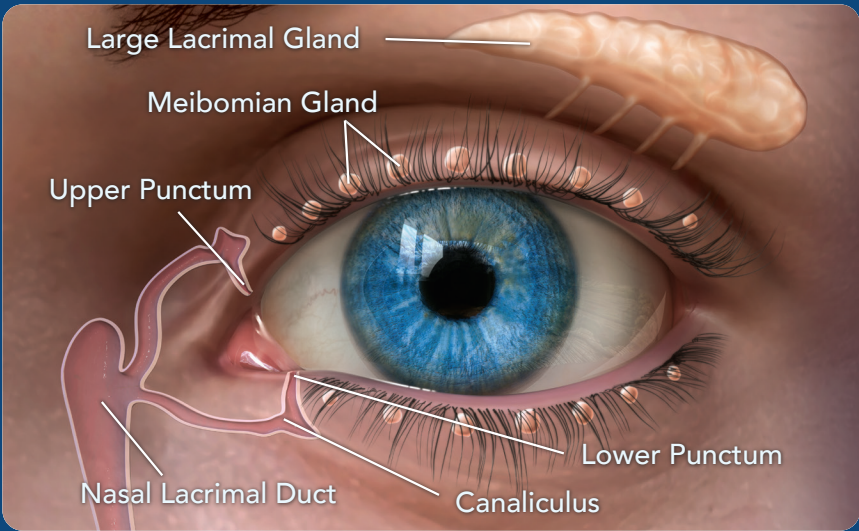


Dry Eye Syndrome is a common condition where your eyes may feel dry, gritty, scratchy, itchy, burning or even feel watery. There are different treatments for Dry Eye. Ask your doctor what treatment is best for your condition.

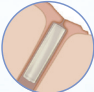


Up to 5 days  
SOFT PLUG®  
COLLAGEN  
PLUGS

Short-Term




Up to 3 months  
SOFT PLUG®  
EXTENDED  
DURATION




Up to 6 months  
SOFT PLUG®  
EXTENDED  
DURATION 180


Medium-Term




Up to 6 months  
SOFT PLUG®  
EXTENDED  
DURATION  
180-T TAPERED



FORM FIT®  
HYDROGEL  
PLUGS



SOFT PLUG®  
SILICONE  
PLUGS



SOFT PLUG®  
SILICONE  
PLUGS  
FLOW  
CONTROL

Long-Term

Nourishing and sustaining the eye-care community-for the long term.

OASIS® SOFT PLUG®, FORM FIT™, and  
Oasis names and logos are registered trademarks of OASIS® Medical, Inc.  
2035 E Financial Way, Glendora, CA 91741. LIT-OV-PP-WC-S Rev0 05-2025



LIT-OV-PP-WC-S

MARKETING MATERIALS

## WALL CHARTS

REF #	NAME
LIT-DES-WC	Dry Eye Symptoms Wall Chart
LIT-OV-PP-WC-S	Punctal Plug Wall Chart



## ORDERING INFORMATION

Orders can be placed 24 hours a day via our website, fax, or the phone message center.

CUSTOMER SERVICE HOURS  
7:00 AM - 4:30 PM (PST),  
Monday - Friday, Excluding holidays.

USA: 844.820.8940  
800.631.7210 FAX

PAYMENT TERMS: NET 30 DAYS  
SHIPPING: FOB GLENDORA, CALIFORNIA  
Insurance and shipping charges added to invoice.

INTERNATIONAL: 909.305.5400  
909.305.9987 FAX  
international@oasismedical.com

WEB (US ONLY): oasismedical.com

### RETURN MERCHANDISE AUTHORIZATION (RMA)

Unopened items returned for credit must be accompanied by an RGA (return goods authorization number) and the reason for the return. An RMA can be requested through customer service.

ALLOWANCE SCHEDULE FOR ACCEPTABLE RETURNS	
Sellable merchandise returned in factory condition within 60 days	Full Credit
Sellable merchandise returned in factory condition within 60-90 days	Cost less 40%
Sellable merchandise returned in factory condition 90 days to 1 year	Cost less 60%
Merchandise returned one year after invoice date	No Credit
Merchandise unsuitable for resale	No Credit
Merchandise defective at time of receipt by customer	Full Credit
Merchandise shipped in error	Full Credit

Merchandise that will not be accepted:

- "Special Order" or "Custom Products"
- Merchandise not in standard OASIS® packaging
- Opened products
- Discontinued products

*NOTE: Customer must prepay all freight charges on stock returns except for defective material or merchandise shipped in error.*

### DOMESTIC SHIPPING POLICY

Shipments from OASIS® are FedEx only. Choices are Ground, 3-Day Select, 2-Day, Priority Overnight (by 10:30am) or Standard Overnight (by 4:30pm). Shipping costs will be added when order is placed. Customer's FedEx or UPS account number may be used if provided to Customer Service at time order is placed. We cannot ship to PO boxes. We cannot be responsible for delays in transit. In-stock product orders received before 2pm PST Monday-Friday, excluding holidays, will usually ship on the same business day. Prepayment orders will ship once payment is received. Custom orders will require additional shipment times and will ship when the product is available.

### WEBSITE ORDERS








Visit **[oasismedical.com](http://oasismedical.com)** to see terms of use, privacy policy, and returns authorization by web.






## STERILIZATION METHOD

Products are sterilized by ethylene oxide, gamma radiation, electron beam, or steam according to EN 556 (European) guidelines and ANSI/AAMI/ISO 11135, 11137 or 17665 standards for medical device sterilization as applicable, to a sterility assurance level (SAL) of  $10^{-6}$ .








## PRODUCT LABELING

OASIS® uses internationally recognized symbols on its product labeling.

	European Union CE Mark and notified body ID number
	Consult instructions for use.
<b>Rx Label</b>	Physician use only
	Do not reuse, single use
<b>LOT</b> LSmmyyA	Lot or batch code
<b>MD</b>	Medical Device
 yyyy-mm-dd	Use by. Placed next to Expiration Date – year/month
<b>REF</b> [Part No.]	Reference Catalog Number (Numbers/Letter represent part number i.e. 6612D)
<b>EC</b> <b>REP</b>	European Authorized Representative
	Manufacturer
<b>UDI</b>	UDI
	Patient Information Website
	Contains biological material of animal origin
<b>CH</b> <b>REP</b>	Swiss Authorized Representative

	Do not resterilize
	Do not use if package is damaged
	No Latex
	Keep dry
	Temperature limit

## METHODS OF STERILIZATION

	Gamma or E-beam Radiation, Single Sterile Barrier System
	Gamma or E-beam Radiation, Double Sterile Barrier System
	Gamma or E-beam Radiation, Single Sterile Barrier System with Protective Packaging Inside
	Gamma or E-beam Radiation, Single Sterile Barrier System with Protective Packaging Outside
	Ethylene Oxide, Single Sterile Barrier System
	Ethylene Oxide, Double Sterile Barrier System
	Steam or Dry Heat, Single Sterile Barrier System



0297



Donawa Lifescience Consulting Srl  
Piazza Albania 10  
00153 Rome, Italy



MedEnvoy Switzerland  
Gotthardstrasse 28 6302 Zug  
Switzerland



2035 E Financial Way  
Glendora, CA 91741  
LIT-USDE-CAT Rev16 08-2025