**SOFT PLUG®**  
 INTRACANALICULAR PLUG  
**COLLAGEN**
**DESCRIPTION:**

The OASIS Medical Collagen Intracanalicular Plug is designed to be inserted through the punctal opening into the canaliculus in order to block tear drainage through the lacrimal drainage system. The Collagen Intracanalicular Plugs are made from an absorbable suture material. The plugs are 2 mm long and available in multiple sizes: SOFT PLUG® COLLAGEN plugs are available in 0.2 mm diameter, 0.3 mm diameter, and 0.4 mm diameter. There are six plugs per package so that a plug can be placed in each canaliculus and two are left over in case a plug is dropped or mishandled.

Prior to insertion of a Collagen Intracanalicular Plug, inspect the patient's punctum to determine the appropriate size plug to use. Remove the appropriate diameter plug from the foam plug holder using jewelers forceps. Insert the plug part way through the punctal opening while grasping the plug with the forceps. Release the plug and use the tip of the forceps to push the plug the rest of the way through the punctal opening and into the canaliculus. A small amount of lubricant may be used on the plug in order to facilitate insertion across the punctum.

**INDICATIONS FOR USE:**

The OASIS Medical Collagen Intracanalicular Plug is intended for temporary use in patients experiencing dry eye symptoms such as redness, burning, reflex tearing, itching or foreign body sensations which can be relieved by blockage of the canaliculus.

Collagen Intracanalicular Plugs may be used

in the treatment of dry eye syndrome and the dry eye components of various ocular surface diseases such as corneal ulcers, conjunctivitis, pterygium, blepharitis, keratitis, red lid margins, recurrent chalazions, recurrent corneal erosion, filamentary keratitis and other external eye diseases.

When indicated the Collagen Intracanalicular Plug may be used after surgery on the eye to prevent complications due to dry eye and to enhance the retention of ocular medications on the eye.

**CONTRAINDICATIONS:**

The OASIS Medical Collagen Intracanalicular Plug is contraindicated for patients with known sensitivities or allergies to bovine collagen or for patients who are experiencing epiphora, inflammation of the eye lid, or tearing secondary to dacryocystitis with mucopurulent discharge.

If the patient experiences irritation, infection,

or epiphora after insertion of a Collagen Intracanalicular Plug, saline irrigation or probing may be used to expel the plug through the lacrimal sac into the nose or throat. However, these symptoms should be resolved when the implant dissolves.

**CAUTIONS:**

Care should be used not to perforate the punctum or canaliculus while inserting the Collagen Intracanalicular Plug. Perforation may cause pain and increase the risk of infection. If perforation occurs, delay the placement of the plug until the wound heals.

The OASIS Medical Collagen Intracanalicular Plug may enhance the effect of ocular medication used on the eye. Depending on the type of medications being used, dosage may need to be reduced accordingly.

Federal law (USA) restricts this device to sale by or on the order of a physician.

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