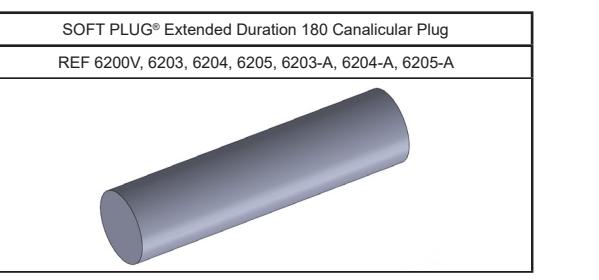
SOFT PLUG® EXTENDED DURATION 180

EN

SOFT PLUG® EXTENDED DURATION 180


BESKRIVELSE:

OASIS Medical SOFT PLUG® Extended Duration 180 Canicular Plug er designet til at blive indsat gennem den punktale åbning i canaliculus for at blokere tredrænagesystemet i ca. 180 dage.

DESCRIPTION:

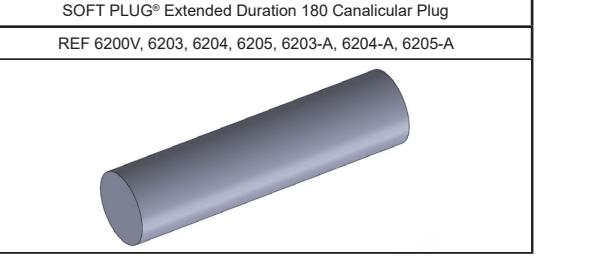The OASIS Medical SOFT PLUG® Extended Duration 180 Canicular Plug is designed to be inserted through the punctal opening into the canaliculus in order to block tear drainage through the lacrimal drainage system for approximately 180 days.

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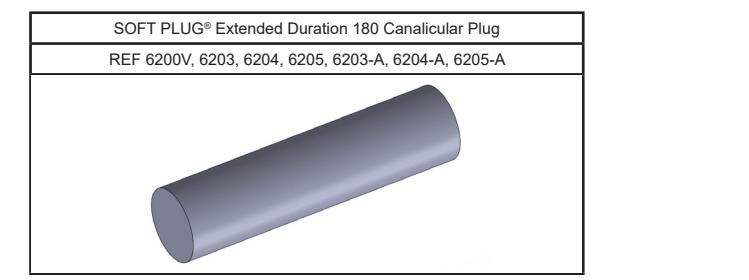
SOFT PLUG® Extended Duration 180 Canicular Plugs are made from degradable polydioxanone monofilament colored violet with D&C No. 2. The plugs are 2.0 mm long and available in 0.3mm, 0.4mm and 0.5mm diameters. The plugs are supplied sterile, two plugs per package, so that a plug can be placed into each canaliculus.

SOFT PLUG® Extended Duration 180 Canicular Plug REF 6200V, 6203, 6204, 6205, 6203-A, 6204-A, 6205-A

SOFT PLUG® Extended Duration 180 Canicular Plugs are made from degradable polydioxanone monofilament colored violet with D&C No. 2. The plugs are 2.0 mm long and available in 0.3mm, 0.4mm and 0.5mm diameters. The plugs are supplied sterile, two plugs per package, so that a plug can be placed into each canaliculus.

SOFT PLUG® Extended Duration 180 Canicular Plug REF 6200V, 6203, 6204, 6205, 6203-A, 6204-A, 6205-A CE-marked.

INDICATIONS FOR USE:

The SOFT PLUG® Extended Duration 180 Canicular Plugs are intended to temporarily block tear drainage by the occlusion of the canaliculus in order to:

- Temporarily treat dry eye syndrome, and the eye dry components of various ocular surface diseases,
- Temporarily enhance the efficacy of topical medications or ocular lubricants,
- Temporarily treat lens intolerance secondary to dry eye,
- Temporarily treat dry eye after ocular surgery, and
- Determine the potential effectiveness of permanent occlusion.

CONTRAINDICATIONS:

The SOFT PLUG® Extended Duration 180 Canicular Plugs are contraindicated for:

- Patients with known sensitivities or allergies to polydioxanone,
- Patients who are experiencing epiphora, canaliculitis, infectious conjunctivitis, or dacryocystitis with mucopurulent discharge.

INSERTION:

Prior to insertion, inspect the patient's punctum to determine the appropriate diameter plug to use. Punctal sizing instruments are available from OASIS Medical.

Apply a topical anesthetic to the eyelid prior to insertion of the plug.

Open the sterile Tyvek® and foil pouches and remove the appropriate diameter plug from the foam holder using sterile jeweler's forceps.

A small amount of lubricant may be used on the plug or punctum in order to facilitate insertion across the punctum.

Insert the plug part way through the punctal opening while grasping the plug with the forceps. Release the plug and use the tip of the forceps to push the plug rest of the way through the punctal opening and into the canaliculus.

Following insertion, inspect the punctal opening to ensure the plug is below the punctal opening.

POTENTIAL RISKS / ADVERSE REACTIONS OF IMPLANTATION:

- Teasing of the punctum or puncture of the canaliculus during insertion,
- Infection related to insertion,
- Excessive epiphora,
- Infection related to contamination trapped in the canaliculus due to insertion of a plug,
- Allergic reaction to polydioxanone resulting in irritation, itching, or swelling,
- Possible need to surgically remove a plug that cannot be flushed out of the canaliculus.

WARNINGS:

DO NOT USE IF PACKAGE IS OPEN OR DAMAGED. Sterility may be compromised or moisture may shorten the shelf life of the plug.

DO NOT REUSE. This is a single use disposable device that cannot be properly cleaned and re-sterilized.

Canalicular plugs may enhance the effect of ocular medications used on the eye. Depending on the type of medications being used, dosage may need to be reduced accordingly.

Should use not be to tear the punctum or puncture the canaliculus while inserting the plug. Perforations may cause pain and increase the risk of infection or perforation occurs, delay the insertion of the plug until the wound heals.

If the patient experiences irritation, infection, or epiphora after insertion, the plug should be removed. Saline irrigation may be used to expel the plug through the lacrimal drainage system into the nose or throat.

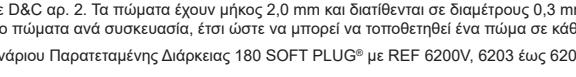
CAUTION:

Federal law (USA) restricts this device to sale by or on the order of a physician.

The Summary of Safety and Clinical Performance (SSCP) is available to the public via European database – EUDAMED.

STORAGE:

Store between 0°C and 35°C (32°F and 95°F). Keep dry.

SOFT PLUG® Extended Duration 180 Canicular Plugs are made from degradable polydioxanone monofilament colored violet with D&C No. 2. The plugs are supplied sterile, two plugs per package, so that a plug can be placed into each canaliculus.

SOFT PLUG® Extended Duration 180 Canicular Plug REF 6200V, 6203, 6204, 6205, 6203-A, 6204-A, 6205-A

SOFT PLUG® Extended Duration 180 Canicular Plugs are made from degradable polydioxanone monofilament colored violet with D&C No. 2. The plugs are 2.0 mm long and available in 0.3mm, 0.4mm and 0.5mm diameters. The plugs are supplied sterile, two plugs per package, so that a plug can be placed into each canaliculus.

SOFT PLUG® Extended Duration 180 Canicular Plug REF 6200V, 6203, 6204, 6205, 6203-A, 6204-A, 6205-A CE-marked.

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