**DESCRIPTION:**

The OASIS Medical SOFT PLUG® Extended Duration is a short term plug (less than 3 months based on results obtained in an animal study) designed to be inserted through the punctal opening into the canaliculus in order to block tear drainage through the lacrimal drainage system. This residency time fails between the absorbable collagen plugs, which are completely absorbed after several days, and the permanent intracanalicular plug, which is designed to remain in the canaliculus indefinitely.

SOFT PLUG® Extended Duration are made from an absorbable copolymer of glycolic acid and trimethylene carbonate and dyed with D&C Green Number 6. They are available in four diameters. The plugs are supplied sterile, two plugs per package, so that a plug can be placed into each canaliculus.

**INDICATIONS FOR USE:**

The SOFT PLUG® Extended Duration is intended for temporary use in patients experiencing dry eye symptoms such as redness, burning, reflex tearing, itching, or foreign body sensations which can be relieved by blockage of the canaliculus. They may be used in the treatment of dry eye syndrome and the dry eye components of various ocular surface diseases.

When indicated, the SOFT PLUG® Extended Duration may also be used after surgery on the eye to prevent complications due to dry eye and to enhance the retention of ocular medication on the eye.

**CONTRAINDICATIONS:**

The OASIS Medical SOFT PLUG® Extended Duration is contraindicated for patients with known sensitivities or allergies to glycolic acid or trimethylene carbonate and for patients who are experiencing epiphora, inflammation of the eye lid, or tearing secondary to dacryocystitis with mucopurulent discharge.

The plug is not intended for prophylactic use and should only be prescribed if dry eye symptoms are present.

**INSERTION:**

Prior to insertion of a SOFT PLUG® Extended Duration, inspect the patient's punctum to determine the appropriate diameter plug to use. Punctal sizing instruments are available from OASIS Medical.

Open the sterile Tyvek pouch and remove the appropriate diameter plug from the blue foam holder using jeweler's forceps.

A small amount of lubricant may be used on the plug or punctum in order to facilitate insertion across the punctum.

Insert the SOFT PLUG® Extended Duration part way through the punctal opening while grasping the plug with the forceps. Release the plug and use the tip of the forceps to push the rest of the way through the punctal opening and into the canaliculus.

**CAUTION:**

The OASIS Medical SOFT PLUG® Extended Duration may enhance the effect of ocular medication used on the eye. Depend-ing on the type of medications being used, dosage may need to be reduced accordingly.

Care should be used not to perforate the punctum or canaliculus while inserting the SOFT PLUG® Extended Duration. Perforations may cause pain and increase the risk of infection. If perforation occurs, delay the placement of the plug until the wound heals.

If patient experiences irritation, infection, or epiphora after insertion of a SOFT PLUG® Extended Duration, the plug should be removed. Saline irrigation or probing may be used to expel the plug through the lacrimal drainage system into the nose or throat.

**DO NOT REUSE:** This is a single use, disposable device that cannot be properly cleaned and re-sterilized.

Federal law (USA) restricts this device to sale by or on the order of a physician.

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