PUNCTUM PLUG
SILICONE

INTENDED USE:
The OASIS® Medical SOFT PLUG® Silicone Punctum Plugs are intended to be inserted into the punctal opening by a certified physician to totally or partially block tear drainage through the canaliculus.

DESCRIPTION:
The OASIS® Medical SOFT PLUG® Silicone and Flow Control Punctum Plugs are made from medical grade silicone and are available in two styles: the original, OASIS® Medical SOFT PLUG® Silicone, with a closed head portion which blocks flow through the punctal opening, and the SOFT PLUG® Flow Control, with an opening through the nose portion which allows flow through the punctal opening.

Product Ref Code Product Trade Name
66xx SOFT PLUG® Silicone Punctum Plug
66xx-FC SOFT PLUG® Flow Control Plug

The OASIS® Medical SOFT PLUG® Silicone Punctum Plug has a thin cap portion designed to remain outside the punctum and rest on the eyelid. The head portion of the plug is conical with a larger diameter designed to fit in the canaliculus to anchor the plug in the punctum. The central shaft portion of the plug has a smaller diameter than the head and cap portions and is designed to fit through the punctal opening.

In practice, the plug insertion procedure is generally recognized as a non-sterile procedure. Oasis Medical sterilized its plugs to provide the best quality product possible.

ANESTHESIA:
Anesthesia area of the punctum with either a topical anesthetic instilled into the conjunctival sac or an anesthetic soaked, cotton-tipped applicator held against the punctum for 30 seconds.

PROPER SIZING:
Determine the correct plug size prior to planned dilation. Start with the smallest gauge size (5 mm). Gently insert the head of the gauge into the punctum, being careful not to go beyond the neck. When entering and leaving the punctal opening, the gauge should slide past the punctal ring. If there is no resistance to the gauge, the next larger size should be tried. The gauge size is too large if there is substantial resistance and the tip is difficult to insert. The proper plug size is found when the punctal opening feels moderately when the gauge is inserted and removed and the gauge is "swung" when in the punctum. In the punctum, the left and right eyes may have different size punctum, therefore it is useful that the punctum of both eyes are measured.

DILATION (OPTIONAL):
If dilation, gently dilate the punctum using a commonly available dilator. Once dilated, the silicone plug needs to be quickly inserted. If the plug is not inserted within 15 to 30 seconds, the dilation may need to be repeated.

THE OASIS® MEDICAL SOFT PLUG®: Silicone Punctum Plugs come preloaded on a disposable inserter. Use the inserter to repeat the plug across the punctum. Lubrication of the plug with saline solution or artificial tears eases insertion. Grasp the inserter on the non-compressible side to avoid compressing the plug out of the eye. Gently squeeze and withdraw the inserter to release the plug. Observe and verify proper placement of the punctum plug. The nose of the plug should be beyond the punctum, the shaft of the plug should be through the punctum, and the cap of the plug should be outside the punctum and resting snug against the eyelid. If provided insertion instructions are not followed, it can result in patient injury.

DILATION (OPTIONAL):
If dilation, gently dilate the punctum using a commonly available dilator. Once dilated, the silicone plug needs to be quickly inserted. If the plug is not inserted within 15 to 30 seconds, the dilation may need to be repeated.

THE OASIS® MEDICAL SOFT PLUG®: Silicone Punctum Plugs come preloaded on a disposable inserter. Use the inserter to repeat the plug across the punctum. Lubrication of the plug with saline solution or artificial tears eases insertion. Grasp the inserter on the non-compressible side to avoid compressing the plug out of the eye. Gently squeeze and withdraw the inserter to release the plug. Observe and verify proper placement of the punctum plug. The nose of the plug should be beyond the punctum, the shaft of the plug should be through the punctum, and the cap of the plug should be outside the punctum and resting snug against the eyelid. If provided insertion instructions are not followed, it can result in patient injury.

INJECTION:
The OASIS® Medical SOFT PLUG® Silicone Punctum Plugs are indicated for use in patients experiencing dry eye symptoms such as redness, burning, reflex tearing, itching or irritation. The OASIS® Medical SOFT PLUG® Silicone Punctum Plugs are indicated for use in patients experiencing dry eye symptoms such as redness, burning, reflex tearing, itching or irritation.

INDICATIONS FOR USE:
The OASIS® Medical SOFT PLUG® Silicone Punctum Plugs are indicated for use in patients experiencing dry eye symptoms such as redness, burning, reflex tearing, itching or irritation. The OASIS® Medical SOFT PLUG® Silicone Punctum Plugs are indicated for use in patients experiencing dry eye symptoms such as redness, burning, reflex tearing, itching or irritation.

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